

Finishing & Polishing Products

for Mollosil®plus and Molloplast-B®



The successful completion of high performance soft relines requires the use of the correct set of instruments and polishes, in the correct sequence, to obtain the highest quality results. The following products from Buffalo Dental, used in the suggested steps, produce the optimum final surface quality in the shortest time for Mollosil plus and Molloplast-B relines.

Step 1 – Reduction: SoftGrind™ Grinding Sleeves

After removing gross amounts of excess material with scissors or a sharp blade, SoftGrind Grinding Sleeves are the ideal instruments for all-purpose trimming and finishing of the surfaces of soft reline materials. The 5/16" OD x 1/2" Long friction-fit grinding sleeves with angled tip are specially designed to grind quickly and smoothly without clogging, and may be disposed after use. The reusable rubber-tipped sleeve-holder is mounted on a convenient 3/32" diameter straight HP mandrel (with 1 1/2" gripping length) and is easy and convenient to use.



<u>Item #</u>	<u>Description</u>
---------------	--------------------

62400	SoftGrind Intro Kit, Pkg. of 10 Coarse Grinding Sleeves and 1 Reusable HP Mandrel
-------	---

62410	SoftGrind HP Mandrel Refill, Pkg. of 1
-------	--

62420	SoftGrind Grinding Sleeves Refill, Pkg. of 10
-------	---

Step 2 – Finishing: Pre-Polishers

Pre-Polishers are the ideal instruments for smoothing trimmed surfaces in preparation for final polishing. Use of the honeycomb-textured Pre-Polishers smooths the silicone surface and removes small amounts of material without gouging or heating. Wheels are 1 1/8" Diameter x 1/4" Thick.



62200	Pre-Polisher Kit, Pkg. of 10 Pre-Polisher Wheels and 1 Reusable HP Mandrel
-------	--

Also available-

For one-step trimming and gross reduction of Mollosil *plus* and Molloplast-B surfaces are the original **MOLLOCUTTER Steel HP Burs** in an assorted pkg. of 3 (#62280).

And for general grinding and smoothing of relined surfaces, request our original **Mollobrasive HP Stone Points** in an assorted package of 3 (#62290).



Mollocutter Burs



Mollobrasive Points

Buffalo Dental Manufacturing Co., Inc.,

159 Lafayette Dr., PO Box 678, Syosset, NY 11791-0678 | Tel: 516-496-7200 or 800-828-0203

Fax: 516-496-7751 | Email: Sales@mailbdm.com | www.buffalodental.com

S62400 0418